



HUBER INFUSION SET

To Be Used With Implanted Vascular Access Ports

Safety Vascular Access Non-Coring Right Angle Needle and Administration Set with On/Off Clamp

Directions: *Use Aseptic Technique*

INSERTION

1. Grasp the wings between thumb and forefinger of dominant hand. (Fig.1). Remove needle guard with non-dominant hand locate and stabilize the vascular access port.
2. Using the forefinger and middle finger of the non-dominant hand locate and stabilize the vascular access port.
3. Position the needle over port and insert needle perpendicular into vascular access port by pressing down with index finger while aiming for the center of the port septum. Advance needle through the skin and septum until reaching the bottom of reservoir (Fig.2).
4. Secure per Institution Protocol.
5. Firmly attach female adapter to tubing.
6. Fill port in accordance with institution protocol.
7. Port manufacturers generally recommend not using syringes less than 10cc.

To De-Access or Remove Safety Needle

8. Position forefinger and middle finger of non-dominant hand on base (Fig.3).
9. Firmly grasp wings between thumb and forefinger of the dominant hand, remove the needle from the port (Fig.3.). Fully retract the needle to ensure the needle point is positioned within the protective well.
10. Using the thumb and forefinger of the dominant hand snap the locking mechanism in place (Fig.4.) USER WILL SEE AND FEEL LOCKING MECHANISM LOCK IN PLACE.

CAUTIONS:

- Leakage may occur when disconnecting components.
- Place in sharps container.
- Change set according to CDC guidelines
- Inspect the package and the product. Do not use if package is damaged or opened.
- Single use product- DO NOT REUSE OR RESTERILIZE.
- RX Only

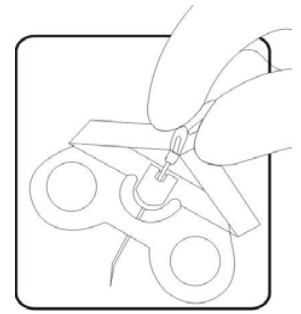


Fig.1

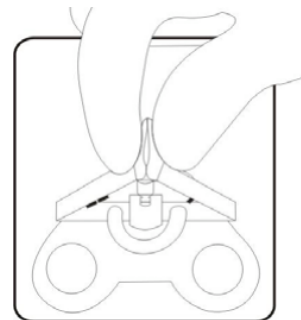


Fig.2

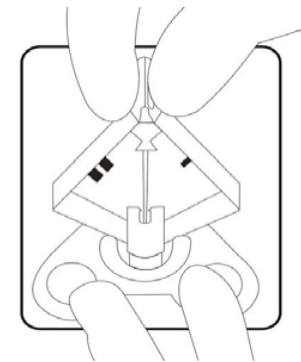


Fig.3

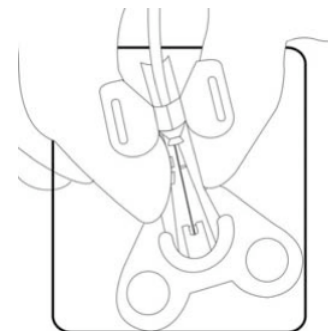


Fig.4